



[www.MedAid.ie](http://www.MedAid.ie)

**CE** This product conforms to the Medical Device Regulations 2017/745

## MEDEVAC4 STRETCHER – SETUP INSTRUCTIONS

**STEP 1 – Release the tie straps.**



**STEP 2 – Straighten stretcher collapsible support rods.**





**STEP 3 – Hold both handles and rotate 90° outwards.**



**STEP 4 – While holding handles, push against cross rod and lock into place. Turn stretcher around and lock second rod into place.**



**STEP 5 – Fold stretcher by repeating same actions in reverse. Rotate (turn) handles inwards, fold joints and tie straps.**